oxmed




Call us friendly

Hydrocolloid Paste

REF

A2001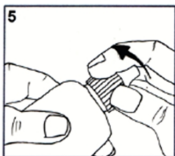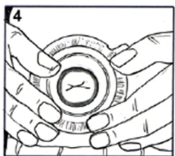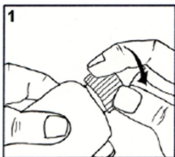

GB - Instructions for use

Features and Intended Use

ZenSetiv Hydrocolloid Paste is a filler and skin barrier that has been specifically designed for use around stomas and fistulas in order to fill skin crevices and create a protective barrier for the skin against harsh output.

Materials/Ingredients

Polyisobutylene,
carboxymethylcellulose, pectin.

Application

- Clean and dry the skin area around the stoma/fistula.
- Apply the paste directly to skin folds, uneven surfaces or scarring.
- Use wet fingers when moulding the paste into place.

Disposal of the device after use

Remove the product and dispose of it in the trash. Do not flush your used product down the toilet because this will block your sewage system.

Warnings and Precautions

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.
- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.
- Never apply to an open wound.
- Replace cap on tube after use.

Contraindications

These devices should not be used by anyone with a known sensitivity to the materials or Ingredients listed in the section 'Materials/Ingredients'.

Storage Conditions

It is best to store your products in a cool dry place with a guidance of temperature from 0°C to 27°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf Life

Use by the expiry date noted on the label.

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Materialen/Ingrediënten

Polyisobutyleen,
carboxymethylcellulose, pectine.

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Oxmed International GmbH