oxmed

Call us friendly




Hydrocolloid Half Moon Strips

REF

A2015

A2015-6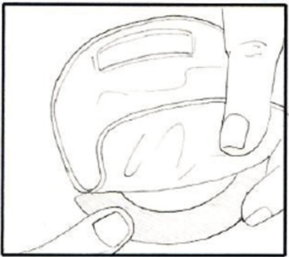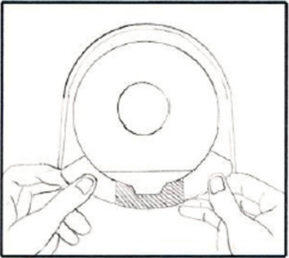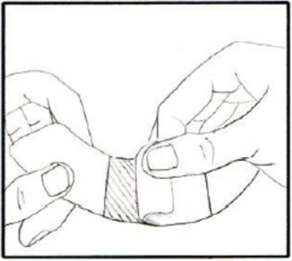

GB - Instructions for use

Features and Intended Use

ZenSetiv Hydrocolloid Half Moon Strips are for use as extra security around ostomy pouch hydrocolloid or two piece base plates.

Materials/Ingredients

Hydrocolloid - Polyisobutylene, carboxymethylcellulose, pectin, glycerine.

Backing Materials – Polyethylene Film and Release Paper.

Application

Use the Half Moon Strips to create an additional area of adhesion. Before attaching the pouch adhere the strip to the base plate in the desired position. Replace as necessary.

Disposal of the device after use

Remove the product and dispose of it in the trash. Do not flush your used product down the toilet because this will block your sewage system.

Warnings and Precautions

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.
- Each product is for single use only. Do not re-use. Re-use can lead to cross-contamination.
- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.

Contraindications

These devices should not be used by anyone with a known sensitivity to the materials or Ingredients listed in the section 'Materials/Ingredients'.

Storage Conditions

It is best to store your products in a cool dry place with a guidance of temperature from 0°C to 27°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf Life

Use by the expiry date noted on the label.

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Oxmed International GmbH