oxmed

Call us friendly

# Absorbent Sachets

**REF**

**A2020**

**A2020-6**

Do not swallow.

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Nemojte gutati.

Nepolykejte.

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No lo tragues.

Ei saa niellä.

Ne pas avaler.

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## GB - Instructions for use

### **Features and Intended Use**

Highly absorbent granules to thicken the output collected in stoma pouches and to help neutralise odour. The “film layer” surrounding the granules dissolves as soon as it comes into contact with a stool. The highly absorbent granules then release to thicken the stool. The granules will neutralise odours.

### **Materials/Ingredients**

Super absorbent polymers.

### **Application**

- If you use a colostomy pouch (closed), insert one sachet containing absorbent granules, into a new stoma pouch before application.
- If you use an ileostomy pouch (open), empty the pouch and clean the closure, then insert a sachet through the closure of the pouch.

### **Disposal of the device after use**

This product can be used in colostomy pouches and may be disposed of in the same way as you usually dispose of your pouches. When using an ileostomy pouch the contents including the granules may be emptied from the pouch and flushed away in the toilet.

### **Warnings and Precautions**

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.
- Each product is for single use only. Do not re-use. Re-use can lead to cross-contamination.

- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.
- Keep out of reach of children.
- If the contents of the sachet comes into contact with the eyes, rinse thoroughly with clean water and consult a doctor if irritation occurs.
- Do not swallow. If product is swallowed there may be a risk of suffocation. Do not induce vomiting. Rinse mouth with water and seek medical assistance immediately. Monitor until all granules are cleared from mouth and throat.
- Non-toxic.

### **Contraindications**

These devices should not be used by anyone with a known sensitivity to the materials or Ingredients listed in the section 'Materials/Ingredients'.

### **Storage conditions**

Store in a cool dark dry place away from moisture. Best stored with a guidance of temperature from 0°C to 27°C as is noted on the product label.

### **Serious Incidents**

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

### **Shelf Life**

Use by the expiry date noted on the label.

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Oxmed International GmbH