oxmed





Call us friendly

ZenSetiv Stoma Belt

REF

ABS125

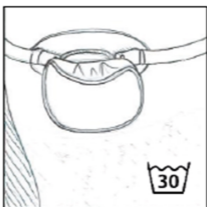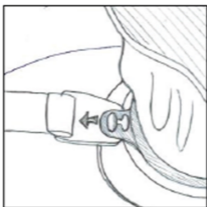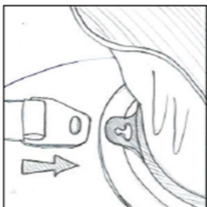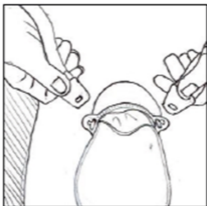GB - Instructions for use

Features and Intended Use

The ZenSetiv Stoma Belt is an elasticated belt used to provide additional security to assist in maintaining a stoma bag in place during wear. It has 2 attachment points that exert even pressure on the base plate. Belt size is 125cm long x 2.8 cm wide and is adjustable.

Materials

Belt: 90% Cotton and 10% man-made fibre which is an elastic to facilitate the belt stretch.

Loops: ABS polymer (acrylonitrile butadiene styrene).

Application

- Attach your ostomy appliance as usual.
- Then, passing the belt around your body, fasten each end of the belt by hooking the lugs of the belt into the eyelets on the belt holder of the ostomy appliance.
- The belt is individually adjustable to abdominal circumference using the adjustment loops.
- Make sure that the belt is not twisted.

Disposal of the device after use

When the product cannot be re-used any more dispose of it in the trash.

Warnings and Precautions

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.

- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.
- Incorrect adjustment to the abdominal circumference can lead to pressure points. Avoid adjusting too tightly.
- The belt can be washed at a low temperature in a laundry bag. A 30°C temperature is recommended.

Contraindications

These devices should not be used by anyone with a known sensitivity to the materials or ingredients listed in the section 'Materials'.

Storage conditions

No special storage conditions required.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf Life

Use by the expiry date noted on the label.

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Oxmed International GmbH