oxmed

Call us friendly

Silicone Plaster

REF



ASP2100

ASP275

ASP2100-2

ASP275-2

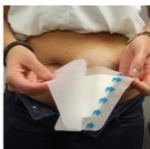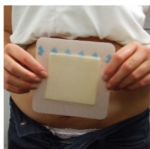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GB - Instructions for use

Features and Intended Use

ZenSetiv Silicone Plasters are non-sterile single-use dressings intended to protect a stoma after irrigation or a continent stoma and may be used with fistulae where the attending physician decides a non-sterile dressing may be applied. The central island material of the plaster will absorb and retain secretion. The silicone-coated polyurethane foam adheres the plaster to the skin to prevent leakage and is non-irritant and easy to remove.

Materials

A plaster of polyurethane film coated with silicone gel with an absorbent central island of polyurethane foam. Overlapping release film is adhered to the plaster for ease of application.

Application

- Clean the skin around the stoma with water if necessary and dry the area well.
- Avoid the use of alcohols, sprays or creams as this could affect the adhesion of the plaster.
- Remove one side of the release film that covers the adhesive.
- Remove the other side of the release film and stick the plaster to the skin.
- Hold the plaster with the adhesive side towards your body.
- Apply the plaster over the stoma from bottom to top.
- Ensure to adhere it evenly to the body and smooth any creases that may occur.
- Secure the plaster by rubbing it gently.

Frequency of replacement of the plaster

Change the plaster when leakage occurs, when the dressing is completely saturated, once a day or when advised by your healthcare professional.

Disposal of the device after use

Remove the product and dispose of it in the trash. Do not flush your used product down the toilet because this will block your sewage system.

Warnings and Precautions

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.
- Each product is for single use only. Do not re-use. Re-use can lead to cross-contamination.
- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.
- When using with a stoma - silicone plaster should only be used after irrigation of the stoma or when the stoma is known to be continent.
- Do not use while swimming or bathing.

Contraindications

These devices should not be used by anyone with a known sensitivity to the materials or ingredients listed in the section 'Materials'.

Storage Conditions

It is best to store your products in a cool dry place with a guidance of temperature from 0°C to 27°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf Life

Use by the expiry date noted on the label.

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Oxmed International GmbH