oxmed

Call us friendly

Hydrocolloid Shapes

REF

HSR090

HSR120

HSR125

HSR200

HSR522

HSU200

HSU311

HSU511

HSF311

HSF511

HSF522

HSR120-2

HSR125-2

HSU200-2

HSU311-2

HSU511-2

GB - Instructions for use

Features and Intended Use

ZenSetiv hydrocolloid shapes are made of skin friendly and absorbent hydrocolloid which is backed with a layer of flexible PE film or soft foam. ZenSetiv hydrocolloid shapes are non-sterile. They can be used in any appropriate circumstances where the attending physician or healthcare professional decides a non-sterile dressing may be applied.

Intended Use

As a skin protector with stomas, fistulas and wounds against aggressive output and against pressure from the hard components of stoma pouches, like urine taps, clamps, girdles etc.

As fixation material to add extra adhesion to the edges of a wound dressing or the skin wafer of stoma appliances.

The round skin wafers can be used as an extra absorption material underneath the daily stoma appliance.

As a skin protector with heavily exudative wounds, protecting the healthy edges of the wound from maceration by exudates or by continuous contact with the saturated absorbent dressing.

As a skin protector where there is a threat of ulceration or blister forming.

Variant

ZenSetiv hydrocolloid shapes are available in three ranges and in different shapes and sizes, as described in the following table:

Regular - Standard hydrocolloid with PE backing

Article code - Size in cm - Shape

HSR090 - 9 Ø - Round

HSR120 - 12 Ø - Round

HSR125 - 12,5x12,5 - Square with rounded edges

HSR200 - 20x20 - Square with rounded edges

HSR522 - 5x22 - Strips with rounded edges

Ultra Thin - Extra thin hydrocolloid with PE backing

Article code - Size in cm - Shape

HSU200 - 20x20 - Square with rounded edges

HSU311 - 3x11 - Strips with rounded edges

HSU511 - 5x11 - Strips with rounded edges

Soft Foam - Standard hydrocolloid with foam backing

Article code - Size in cm - Shape

HSF311 - 3x11 - Strips with rounded edges

HSF511 - 5x11 - Strips with rounded edges

HSF522 - 5x22 - Strips with rounded edges

Materials/Ingredients

Hydrocolloid - Polyisobutylene, carboxymethylcellulose, pectin, glycerine.

Backing Materials – Polyethylene Film/Polyethylene Foam and Release Paper.

Application

Preparation: Choose the correct size, form and variant of ZenSetiv hydrocolloid shapes. Cut the shape when needed. The dressing should be at least 1.5 to 2 cm larger all around the area to be covered.

Appliance: Clean the skin with lukewarm water and pat dry. Do not scrub the skin. Apply the dressing to the desired area of skin and press firmly. Caution! If the skin is very wet use the ZenSetiv hydrocolloid powder before applying the dressing.

Removal: Remove the hydrocolloid by 'peeling' it off the skin. Support the skin and remove from top to the bottom, in a single motion. Remove slowly. Do not tear off the hydrocolloid, this can cause skin damage. It might occur that some residue or adhesive layer remains on the skin after removal.

Frequency of replacement of the product

Change the product as advised by your healthcare professional.

Disposal of the device after use

Remove the product and dispose of it in the trash. Do not flush your used product down the toilet because this will block your sewage system.

Warnings and Precautions

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.

- Each product is for single use only. Do not re-use. Re-use can lead to cross-contamination.
- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.
- Product is latex-free.
- Product do not contain ingredients of animal origin.

Contraindications

These devices should not be used by anyone with a known sensitivity to the materials or Ingredients listed in the section 'Materials/Ingredients'.

Storage Conditions

It is best to store your products in a cool dry place with a guidance of temperature from 0°C to 27°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf Life

Use by the expiry date noted on the label.

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Oxmed International GmbH