oxmed

Call us friendly

ZenSetiv Adhesive Remover Spray

REF

ZSA2011

Avoid contact with eyes.

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GB - Instructions for use

Features and Intended Use

ZenSetiv Adhesive Remover Spray is specially formulated and intended for the safe and pain free removal of medical adhesives from the skin. ZenSetiv Adhesive Remover Spray can be used to aid removal of, for example, wound dressings, tapes and stoma flanges and is effective for removing areas of adhesive build up on the skin.

Materials

Hexamethyldisiloxane,
Octamethyltrisiloxane and
Dimethicone.

Application

When removing adhesive dressings:

- Lift up edge of plaster or hydrocolloid and apply ZenSetiv Adhesive Remover under the product while lifting off.
- Peel away the product slowly while continuing to apply more adhesive remover as the product begins to lift.
- After wiping clean the surrounding skin area, a new product can be applied if required.

For removing adhesive residue on the skin:

- Apply adhesive remover to the affected area.
- Allow the solution to remain on the adhesive residue for 1-2 minutes.
- Gently wipe clean the skin area.
- Re-apply for extremely difficult residues.

Disposal of the device after use

When the product cannot be re-used any more dispose of it in the trash.

Warnings and Precautions

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.
- ZenSetiv Adhesive remover in liquid form is flammable.
- Use in a well ventilated area.
- Do not use around flames or sources of ignition.
- Do not use if pack is damaged or open.
- Avoid contact with eyes. If contact is made with the eyes flush with water.
- Do not swallow. If product is swallowed do not induce vomiting. Rinse mouth thoroughly with water and seek medical assistance.
- Avoid breathing vapours or spray and do not eat or drink while using the product.
- Keep out of reach of children.

Contraindications

These devices should not be used by anyone with a known sensitivity to the materials or ingredients listed in the section 'Materials'.

ZenSetiv Adhesive Remover should not be used on skin where infection is present.

Storage conditions

Keep only in the original container in a cool well ventilated place out of direct sunlight. Store between 5°C and 30°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf life

Use by expiry date printed on the base of the container.

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Oxmed International GmbH