



















<b>GB</b>	Instructions for use	<b>NL</b>	Gebruiksaanwijzing	<b>DE</b>	Gebrauchsanweisung
<b>AR</b>	تعليمات الاستخدام	<b>BA</b>	Uputstvo za upotrebu	<b>CZ</b>	Návod k použití
<b>DK</b>	Brugsanvisning	<b>ES</b>	Instrucciones de uso	<b>FI</b>	Käyttöohje
<b>FR</b>	Mode d'emploi	<b>GR</b>	Οδηγίες χρήσης	<b>HR</b>	Upute za uporabu
<b>ID</b>	Petunjuk Penggunaan	<b>LT</b>	Naudojimo instrukcijos	<b>LV</b>	Lietošanas instrukcija
<b>NO</b>	Bruksanvisning	<b>PL</b>	Instrukcja użycia	<b>PT</b>	Instruções de uso
<b>RO</b>	Instructiuni de folosire	<b>SE</b>	Bruksanvisning	<b>SI</b>	Navodila za uporabo
<b>SK</b>	Inštrukcie na používanie	<b>UA</b>	Інструкція із застосування		

## REF

D1NB10	D1NB10-HC	D1NB25	D1NB29X
D1NB32	D1NB38	D1ND10	D1ND10-HC
D1NT10	D1NT10XV	M1NB10	M1NB10X
D1NB10X	D1NB25X	D1NB32X	D1NB38X
D1NB10XV	D1NB10XV-HC	D1NB25XV	D1NB29XV
D1NB32XV	D1NB38XV	D1FB10XV	D1NT44
D1NT51	D1NB10X-5	D1NB10XV-5	D1FB10XV-5
M1NB10X-5	D1NZ10	D1NZ10-5	D1NZ10X
D1NZ10X-5	D1NZ10XV	D1NZ10XV-5	D1NZ25XV
D1NZ25XV-5	D1NZ29XV	D1NZ32XV	D1NZ32XV-5
D1NZ38XV	D1NZ38XV-5	D1FZ10XV	D1FZ10XV-5
D1NZ29XV-5	P1NB10X	D1NB10XV-30	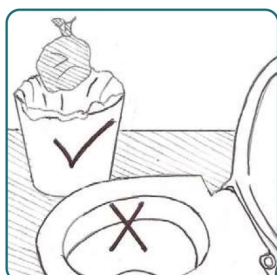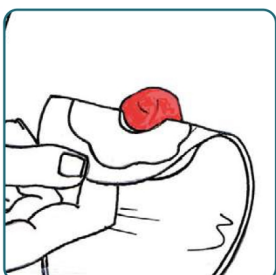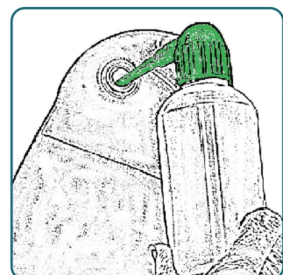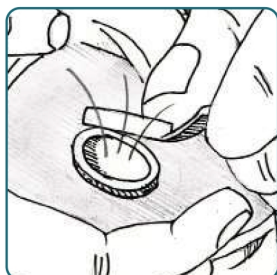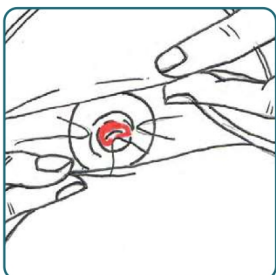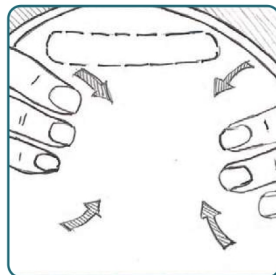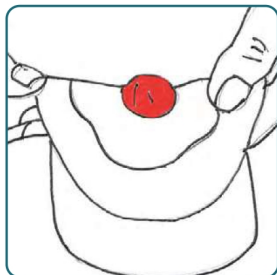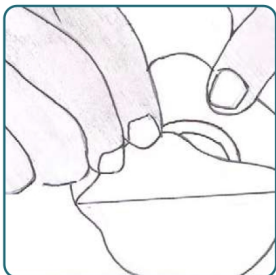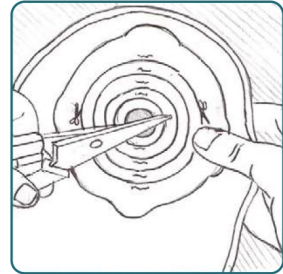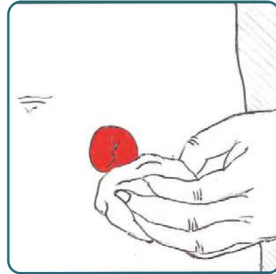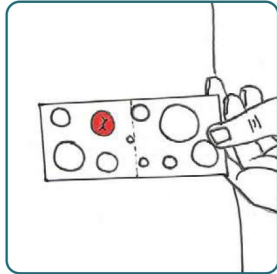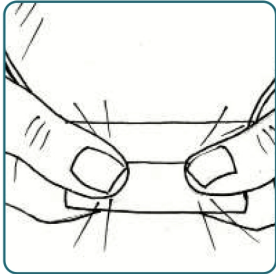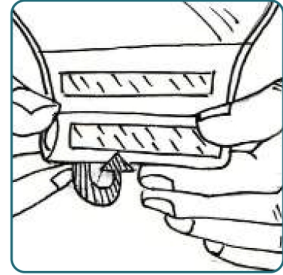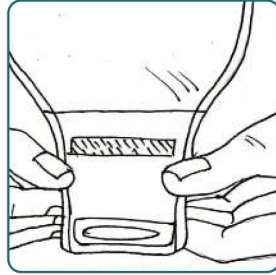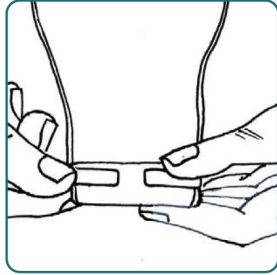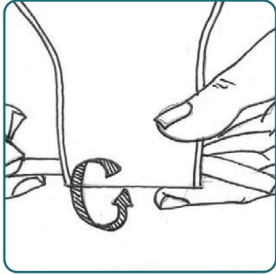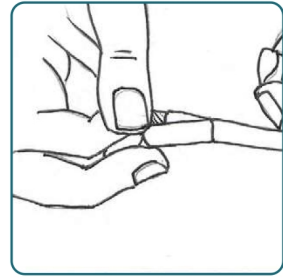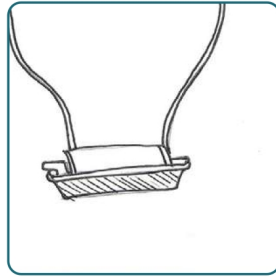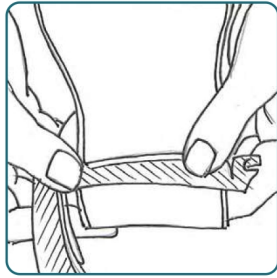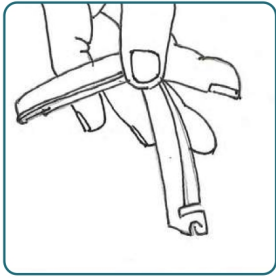

## GB - Instructions for use

### Intended Use, Product Description and Material Information

Ostomy pouches are used to collect waste from the body and are usually attached to the skin over a surgically made stoma.

Zensiv Ostomy products are a range of 1-piece and 2-piece ostomy pouches and 2-piece flanges in various sizes and shapes. All 1-piece pouches and 2-piece flanges feature a skin friendly hydrocolloid wafer for adhesion to the body and this is protected before use with a release film that shows a cutting guide. The product pouch is manufactured from odour barrier film with a comfort non-woven layer and a carbon filter for odour control (for ileostomy and colostomy). Ileostomy pouches have either a roll up drain port secured with an integral 'velcro' type closure or there are clamps and soft ties supplied in each box. For urostomy the pouches carry a soft polymer tap for easy drainage. All pouches have a viewing window to inspect the stoma if needed. Some ranges feature a vent with lid for venting the pouch if required.

All ZenSiv Ostomy products are latex-free and do not contain ingredients of animal origin. The products are non-sterile and can be used in any appropriate circumstances where the attending physician or healthcare professional decides a non-sterile product may be applied.

### Warnings-Precautions

- 
- Each pouch is for single use only. Do not re-use. Re-use can lead to cross-contamination.
  - Use only as directed by your healthcare professional.
  - Read the instructions prior to use.
  - Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the pouches.

### Contra-indications

These devices should not be used by anyone with a known sensitivity to: polyethylene, polyamide, medical charcoal, synthetic polymer, ethylene vinyl acetate, polypropylene, polyester, polyisobutylene, carboxymethylcellulose, pectin or glycerine.

### Storage Conditions

It is best to store your products in a cool dry place with a guidance of temperature from 0°C to 27°C as is noted on the product label.

### Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

### Shelf Life

Use by the expiry date noted on the label.

### Collect your materials

Make sure that all necessary materials are ready for use. Collect a measuring guide, a pair of scissors, a pen, wet and dry gauze and a stoma pouch.

### Choose the right size

Measure your stoma with a measuring guide and cut (if necessary) the flange to the right size of the stoma. Make sure that you choose the right size so that the stoma and surrounding skin will stay healthy. Allow a maximum of 2 mm of space between your stoma and flange for a safe and correct fit. If the size is too big, your faeces will damage the skin. If the size is too small, the flange will damage the stoma. Over time your stoma may change shape or size, so periodically measurements may be required again to prevent any leakage.

### Prepare your skin

Support the skin and remove the flange carefully, do not tear off but push the skin back from the flange, use a wet gauze if necessary. Wash the skin surrounding your stoma with warm and unperfumed water. Do not use anything that contains alcohol, perfume or ether to clean your skin as it can cause damage. Do not rub the skin too hard as this might lead to skin damage. Dry the skin carefully and thoroughly to ensure that your pouch adheres well.

Optional: If your skin is irritated and sore you can use skin barrier cream, hydrocolloid powder or skin barrier wipes. Apply the powder, skin barrier cream or wipe gently to the surrounding skin.

### Close the outlet

First make sure that the clamp or 'velcro' type closure is closed safely.

For the clamp: Place the clamp at the tail of the pouch. Fold the end of the tail once over the clamp and make sure that the film of the tail is smooth to ensure that the outlet tail is in the right position and press the clamp firmly. Clamps are reusable.

For the metal wire strips: Take the release film from the metal wire strip. Stick the metal wire strip onto the tail of the pouch. Roll-up the tail of the pouch. Fold the metal wire on both sides so that the tail of the pouch stays rolled-up.

For the Velcro closure: Roll-up the tail of the pouch until the two 'velcro' type pieces are touching each other. Press the 'velcro' type pieces onto each other firmly.

### Fit your pouch

Remove the release paper of the flange. Fold the flange halfway and place the opening at the underside of the stoma. Stretch the skin of your abdomen with a flat hand and pull the whole flange over the stoma into the right position. Work your way around the stoma so that the hydrocolloid will attach to the skin and make sure that the flange is secure. Place your hand over the flange and make sure that a good seal is created.

### Empty your pouch

Lift up the tail of the pouch and open the bag. Empty the contents of your pouch into the toilet.

### Vent

If the pouch is full of gasses (ballooning) or if there is no air in the pouch (a vacuum) the vent can be used to let air in or out of the pouch. Make sure that the pouch is not full of faeces when opening the vent. If the pouch is full than empty the pouch or change it for a fresh one. The vent can be opened by taking off the lid. Carefully close the lid after venting.

### Viewing window

The ostomy pouch includes a viewing window which makes it possible to inspect the stoma as well as the output from the stoma. It is also very helpful when applying the pouch to make sure it is located correctly over the stoma. Lift up the non-woven of the viewing window for inspection.

### Disposal of your pouch

Remove your pouch and dispose of it in the trash. Do not flush your used pouch down the toilet because this will block your sewage system.

### Frequency of pouch change

Change/empty your pouch as advised by your healthcare professional.

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