






GB	Instructions for use	NL	Gebruiksaanwijzing	DE	Gebrauchsanweisung
AR	تعليمات الاستخدام	BA	Uputstvo za upotrebu	CZ	Návod k použití
DK	Brugsanvisning	ES	Instrucciones de uso	FI	Käyttöohje
FR	Mode d'emploi	GR	Οδηγίες χρήσης	HR	Upute za uporabu
ID	Petunjuk Penggunaan	LT	Naudojimo instrukcijos	LV	Lietošanas instrukcija
NO	Bruksanvisning	PL	Instrukcja użycia	PT	Instruções de uso
RO	Instructiuni de folosire	SE	Bruksanvisning	SI	Navodila za uporabo
SK	Inštrukcie na používanie	UA	Інструкція із застосування		

REF

C2B4	C2B5	C2B6	C2B7
C2B8	C2C6	N2B5	C2B4V
C2B5V	C2B6V	C2B7V	C2B8V
C2B4-6	C2B4V-6	C2B5-6	C2B5V-6
C2B6-6	C2B7-6	C2B7V-6	C2B6V-6
N2B5-6	N2B4	N2B6	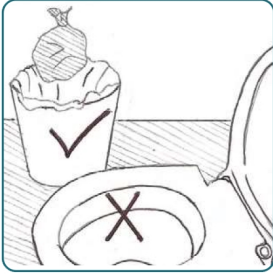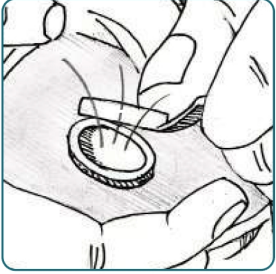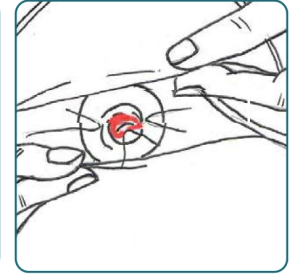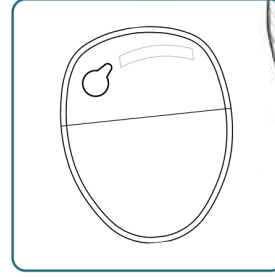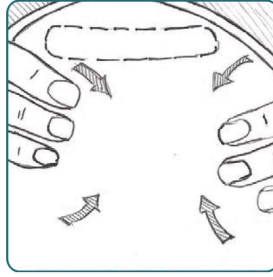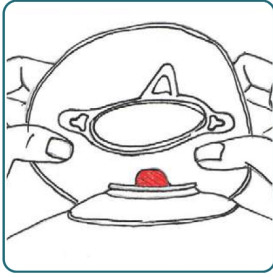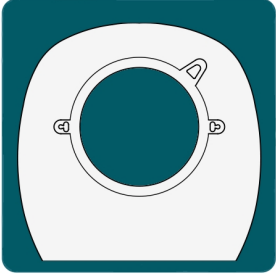

GB - Instructions for use

Intended Use, Product Description and Material Information

Ostomy pouches are used to collect waste from the body and are usually attached to the skin over a surgically made stoma.

Zensiv Ostomy products are a range of 1-piece and 2-piece ostomy pouches and 2-piece flanges in various sizes and shapes. All 1-piece pouches and 2-piece flanges feature a skin friendly hydrocolloid wafer for adhesion to the body and this is protected before use with a release film that shows a cutting guide. The product pouch is manufactured from odour barrier film with a comfort non-woven layer and a carbon filter for odour control (for ileostomy and colostomy). Ileostomy pouches have either a roll up drain port secured with an integral 'velcro' type closure or there are clamps and soft ties supplied in each box. For urostomy the pouches carry a soft polymer tap for easy drainage. All pouches have a viewing window to inspect the stoma if needed. Some ranges feature a vent with lid for venting the pouch if required.

All ZenSiv Ostomy products are latex-free and do not contain ingredients of animal origin. The products are non-sterile and can be used in any appropriate circumstances where the attending physician or healthcare professional decides a non-sterile product may be applied.

Warnings-Precautions

- 
- Each pouch is for single use only. Do not re-use. Re-use can lead to cross-contamination.
 - Use only as directed by your healthcare professional.
 - Read the instructions prior to use.
 - Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the pouches.

Contra-indications

These devices should not be used by anyone with a known sensitivity to: polyethylene, polyamide, medical charcoal, synthetic polymer, ethylene vinyl acetate, polypropylene, polyester, polyisobutylene, carboxymethylcellulose, pectin or glycerine.

Storage Conditions

It is best to store your products in a cool dry place with a guidance of temperature from 0°C to 27°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf Life

Use by the expiry date noted on the label.

Collect your materials

Make sure that all necessary materials are ready for use. Collect a measuring guide, a pair of scissors, a pen, wet and dry gauze and a stoma pouch and flange.

Choose the right size

Measure your stoma with a measuring guide and cut (if necessary) the flange to the right size of the stoma. Make sure that you choose the right size so that the stoma and surrounding skin will stay healthy. Allow a maximum of 2 mm of space between your stoma and flange for a safe and correct fit. If the size is too big, your faeces will damage the skin. If the size is too small, the flange will damage the stoma. Over time your stoma may change shape or size, so periodically measurements may be required again to prevent any leakage.

Prepare your skin

Support the skin and remove the flange carefully, do not tear off but push the skin back from the flange, use a wet gauze if necessary. Wash the skin surrounding your stoma with warm and unperfumed water. Do not use anything that contains alcohol, perfume or ether to clean your skin as it can cause damage. Do not rub the skin too hard as this might lead to skin damage. Dry the skin carefully and thoroughly to ensure that your pouch adheres well.

Optional: If your skin is irritated and sore you can use skin barrier cream, hydrocolloid powder or skin barrier wipes. Apply the powder, skin barrier cream or wipe gently to the surrounding skin.

Fit your base plate

Remove the release paper of the base plate. Stretch the skin of your abdomen with a flat hand and place the base plate over your stoma into the right position (start at the bottom and work your way to the top). Work your fingers around the base plate so that the hydrocolloid bonds well onto the skin and make sure that the plate is secure.

Apply your pouch

Take your pouch and click the pouch onto the ring of base plate. Work your way from the bottom to the top of the ring. Make sure that your pouch is secure and control the complete seal of the pouch with your fingers.

Vent

If the pouch is full of gasses (ballooning) or if there is no air in the pouch (a vacuum) the vent can be used to let air in or out of the pouch. Make sure that the pouch is not full of faeces when opening the vent. If the pouch is full than empty the pouch or change it for a fresh one. The vent can be opened by taking off the lid. Carefully close the lid after venting.

Viewing window

The ostomy pouch includes a viewing window which makes it possible to inspect the stoma as well as the output from the stoma. It is also very helpful when applying the pouch to make sure it is located correctly over the stoma. Lift up the non-woven of the viewing window for inspection.

Disposal of your pouch

Remove your pouch and dispose of it in the trash. Do not flush your used pouch down the toilet because this will block your sewage system.

Frequency of pouch change

Change/empty your pouch as advised by your healthcare professional.

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