
ZenZipp Wound Manager

REF

	Article No.	Description	Zipper	Size mm
	ZZ050030N	Extra Small XS		50x30
	ZZ110075	Small S	X	110x75
	ZZ110075N	Small S		110x75
	ZZ110110	Small Square SQ	X	110x110
	ZZ110110N	Small Square SQ		110x110
	ZZ170120	Medium M	X	170x120
	ZZ220170	Medium Plus MP	X	220x170
	ZZ270190	Large L	X	270x190
	ZZ390240	Extra Large XL	X	390x240
	ZZ170120N	Medium M		170x120
	ZZ220170N	Medium Plus MP		220x170
	ZZ270190N	Large L		270x190

STERILE R

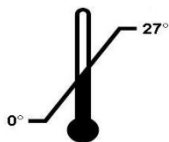1639

EN: Instructions for use

Please read these Instructions before use and follow the guidelines in the Method of use section.

Description

The ZenZipp Wound Manager is a range of pouches in various sizes and shapes designed for wound coverage and drainage. See below for codes and sizes.

- The ZenZipp Wound Manager device is manufactured from odour barrier film and has a non-woven material comfort layer on the body side on all sizes.
- All codes carry a choice of outlet closure. There is either a tap or a roll up Velcro type fastening. One outlet is supplied on the three smaller sizes and three outlets on the Medium to Extra Large sizes.
- All sizes feature a vent with lid for venting the pouch if required.
- The zipper when opened allows easy access to the wound for whatever medical procedures or cleaning are required. Codes with an 'N' have no zipper.
- All sizes feature a skin friendly hydrocolloid wafer for adhesion to the body and this is protected before use with a release paper that shows a cutting guide. The release paper can also be used to make a hand drawn cutting guide, in order to achieve whatever size and shape is needed.

ZenZipp Wound Manager codes and sizes

Article Number	Zipper	Size	Measurements
ZZ050030N	No Zipper	XS	50 x 30 mm
ZZ110075	Zipper	S	110 x 75 mm
ZZ110075N	No Zipper	S	110 x 75 mm
ZZ110110	Zipper	SQ	110 x 110 mm
ZZ110110N	No Zipper	SQ	110 x 110 mm
ZZ170120	Zipper	M	170 x 120 mm
ZZ170120N	No Zipper	M	170 x 120 mm
ZZ220170	Zipper	MP	220 x 170 mm
ZZ220170N	No Zipper	MP	220 x 170 mm
ZZ270190	Zipper	L	270 x 190 mm
ZZ270190N	No Zipper	L	270 x 190 mm
ZZ390240	Zipper	XL	390 x 240 mm

Intended Use

The ZenZipp Wound Manager is for use in draining and collecting output from wounds, stomas, particularly high output difficult to manage stomas and fistulae. Their use where post-operative wounds require drainage and nursing management following surgery allows the healthcare professional to cover the affected area while still being able to drain it of exudate, carry out other care or simply cover over the area during palliative care. The smaller sizes are for use where collection of exudate is required. ZenZipp may be applied by healthcare professionals or by the patient/user following training by their healthcare professional. The ZenZipp has no healing or treatment properties and is only intended for adhesion to intact skin.

Contra-indications

The ZenZipp Wound Manager should be adhered to intact skin and not be used on exposed muscle or bone, so must be cut to fit on the available skin surrounding the wound. It must not be applied to patients who are known to be sensitive to one of the ingredients of the hydrocolloid or materials of the pouch.

Materials and Ingredients:

ZenZipp Pouch Film	Polyamide with PE coating
Non-Woven Layer	Polyethylene
Taps	EVA
Strips	Polyethylene
Vent	EVA
Zipper	Polyethylene based material
Hydrocolloid Wafer	Hydrocolloid backed with EVA and with a polyethylene coated release paper
Hydrocolloid ingredients	Polyisobutylene, carboxymethylcellulose, pectin, glycerine

ZenZipp products are latex-free and do not contain ingredients of animal origin.

Product is packaged in individual Tyvek pouches and boxed.

Product is supplied sterile, sterilised by Gamma Irradiation.

Method of use

1. Use standard procedures for preparing the wound area. The skin surrounding the wound should be clean and dry. Avoid the use of creams and powders, where possible, as this can affect the adhesion of the pouch.
2. Measure the wound and determine which Wound Manager is most suitable.
3. Map a template of the wound area. This can be done by using an acetate or plain paper.
4. With curved scissors, cut the hydrocolloid skin barrier to match the size and shape of the template. Separate areas can be cut into the hydrocolloid for draining fistulae or sinuses.
WARNING: DO NOT cut outside the indicated area.
The cut out pieces of hydrocolloid can be used for extra protection or packing, particularly around the output area of the wound.
5. If necessary, apply a barrier film to the skin surrounding the wound.
6. Remove the release paper and position the pouch over the wound. Smooth into position around the wound; applying gentle pressure. Maintaining this gentle pressure on the hydrocolloid for 30 seconds or so has the effect of maximizing adhesion.
7. If necessary, use a hydrocolloid paste to fill and protect skin folds and unprotected skin between the wound and the skin plate.
8. If necessary, make a barrier using hydrocolloid paste, covering it with moisture-resistant hydrocolloid strips to protect the skin around the wound from aggressive output.
9. If using a Wound Manager with zipper, then close it.
10. The drainage inlet/outlet ports should be closed until a collection device or fluid instillation is required. For drainage, plug the connector from the collection device into the drain port until a tight seal is achieved.
11. The drainage ports can be used as outlets, or inlets for instilling fluids, depending on the clinical circumstances.
12. On Wound Managers that feature a zipper access can be gained to the wound area by opening the zipper, which runs the length of the pouch. Take care to ensure the zipper is fully clipped closed when finished, along the entire length. This will avoid inadvertent leakage and / or odour. Full closure is best achieved by snapping together the two sides at one end of the pouch. Run the two sides up the pouch, between thumb and forefinger. The closure will "zip" together. Ensure the zip mechanism is clean for secure closure.
13. When removing pouch do so carefully using the side of the thumb by working from the outside edge of the hydrocolloid pushing towards the centre all the way round and ensure the skin is not pulled upwards during removal. Use a remover spray if necessary to remove any residue.

Frequency of pouch change

The interval between changes will normally be determined by the degree of exudates produced. The pouch may be left in place for up to 7 days.

Disposal

Following removal dispose of the Wound Manager in hospital contaminated waste or local contaminated waste care disposal.

Useful information

ZenSetiv hydrocolloid paste and hydrocolloid strips can be used to fill skin irregularities. This will create an optimal seal for the hydrocolloid to the skin, preventing leakage and unpleasant smells.

Inspecting the wound at any time without interfering with the healing process is easy to do using either the transparent film on all devices in the range, or opening the zipper on those pouches that feature a zipper.

When opening the pouch on the four larger sizes, the edges of the opening can be fixed on the adjacent Velcro type strips so the bag stays open without the need to hold it, allowing easy access.

Taps allow effective draining, with direct connection to catheter bags, a vacuum pump or other collection devices. The outlets can also be used in a wider form by cutting off the tap and using the Velcro type closure system.

Cautions/Warning

1. Do not use if the pack is torn or damaged as sterility may have been compromised.
2. For single use only. Never re-use this device because cross contamination can occur.
3. The ZenZipp Wound Manager is sterilized by irradiation. Do not re-sterilize this device.
4. Use immediately following opening of the pack.
5. Discontinue use if any allergic reactions occur.
6. If used by an individual - if problems occur with the use of this device contact a healthcare professional for advice.
7. ZenZipp with zipper is not suitable for a patient with very fluid output unless a draining system is in place.

Storage conditions

Store between 0°C and 27°C in a cool dry place. Do not remove the device from the Tyvek pouch until ready to use. Product will not remain sterile if left in an open Tyvek pouch.

Serious Incidents

Should a serious incident arise as a result of using the the sterile ZenZipp Wound Manager, please report this immediately to OxMed International GmbH and the Competent Authority in the country where you reside.

Shelf Life

3 years.

Use by the expiry date noted on the product label.

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