Instructions for Use – GB

Secuthin Skin Barrier Cream 50g

Features and Intended Use

1. The Secuthin Skin Barrier Cream is an aqueous based, white, easy flowing cream without perfume.
2. The Secuthin Skin Barrier Cream is designed for use by ostomy device users to provide a protective layer between the stoma adhesive and the skin.
3. Applied prior to adhesion of the stoma collection device it will not affect the integrity of the hydrocolloid adhesion properties.
4. It is intended to be massaged well into the skin before adhesion of the hydrocolloid baseplate of the stoma appliance.
5. The Secuthin Skin Barrier Cream has been used in conjunction with different types of hydrocolloid and found to be compatible.
6. The Secuthin Skin Barrier Cream contains two common preservatives – Paraben and Phenoxyethanol.

Ingredients

Aqua, Cetearyl Alcohol, Glyceryl Stearate, Ceteareth-25, Isopropylmyristate, Aluminium Chlorhydrate, Paraffinöl, Liquidum, Ceteareth-6, Aluminium/Magnesium-Hydroxide-Stearate, Tocopheryl Acetate, Magnesium/Aluminium Silicate, Panthenol, Propylparaben, Methylparaben, Ethylparaben, Phenoxyethanol.

Application of Secuthin Skin Barrier Cream

1. Remove the lid, remember to always replace the lid securely after dispensing the amount of cream to use.
2. Ensure you use enough cream to coat/cover the area of skin that will be underneath your hydrocolloid baseplate.
3. Ensure to massage the cream well into your skin until the skin feels dry and you can no longer feel any greasiness of the cream.
4. Do not use more than you need – all the cream needs to have soaked into the skin and provide an even layer that coats the surface to be covered by the hydrocolloid baseplate. If you have used too much, wipe it away with a tissue or cloth.
5. Apply the hydrocolloid baseplate of your stoma appliance in the usual way.

Discarding the empty tube

Throw away into a normal waste bin.

Warnings/precautions

1. Read the instructions for use prior to use.
2. For use on intact skin. For external use only.
3. Ensure that the cream is well massaged into the skin.
4. Do not use too much cream as this could affect the adhesiveness of the hydrocolloid baseplate.
5. Discontinue use if you experience any allergic reaction, skin rash or problems when using the Secuthin Skin Barrier Cream and if required, contact a healthcare professional.
6. Do not use with other creams or perfumes as this could affect the adhesiveness of the hydrocolloid baseplate or could cause adverse skin reactions.
7. Always replace the lid back onto the tube after each use.
8. Keep out of reach of children.

Storage conditions

No special storage conditions are required.

Contraindications

1. Do not use if you have a known allergy to any of the materials used in the Secuthin Skin Barrier Cream.
2. Do not use with other creams or perfumes.
3. Poor manual dexterity or with visual impairment - do not use without assistance.

Serious incidents

Should a serious incident arise as a result of using the Secuthin Skin Barrier Cream please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf life

Use by the expiry date noted on the label.

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