It's all
about
ostomy!

























GB	Instructions for use	NL	Gebruiksaanwijzing	DE	Gebrauchsanweisung
AR	تعليمات الاستخدام	BA	Uputstvo za upotrebu	CZ	Návod k použití
DK	Brugsanvisning	ES	Instrucciones de uso	FI	Käyttöohje
FR	Mode d'emploi	GR	Οδηγίες χρήσης	HR	Upute za uporabu
ID	Petunjuk Penggunaan	LT	Naudojimo instrukcijos	LV	Lietošanas instrukcija
NO	Bruksanvisning	PL	Instrukcja użycia	PT	Instruções de uso
RO	Instructiuni de folosire	SE	Bruksanvisning	SI	Navodila za uporabo
SK	Inštrukcie na používanie	TR	Kullanım talimatları	UA	Інструкція із застосування

P1NT10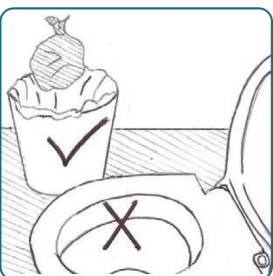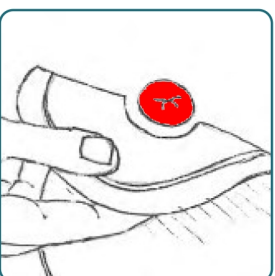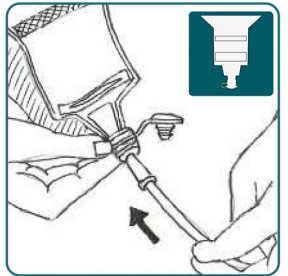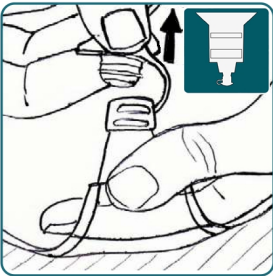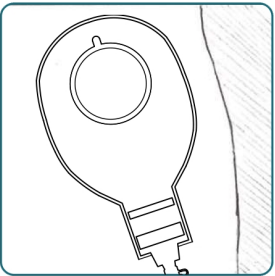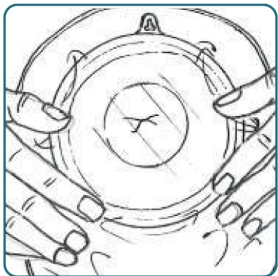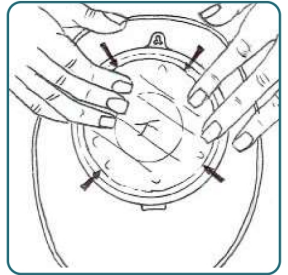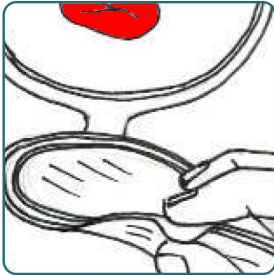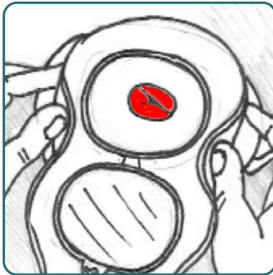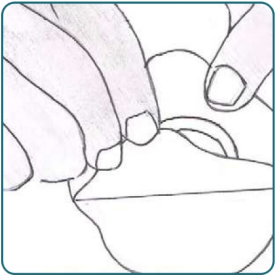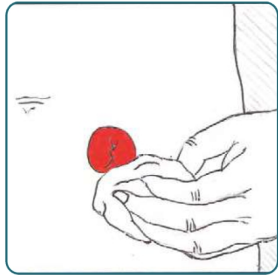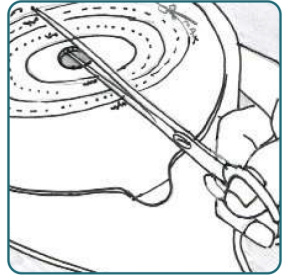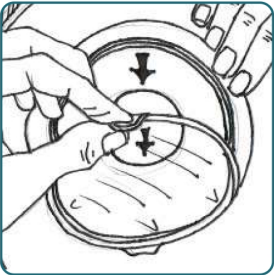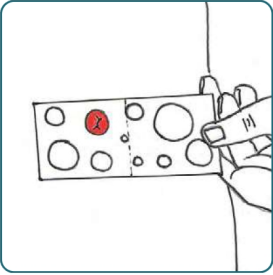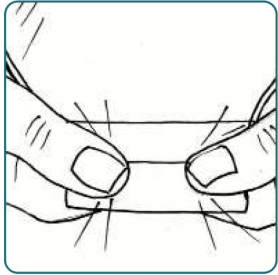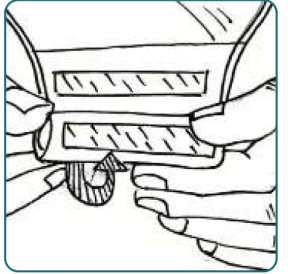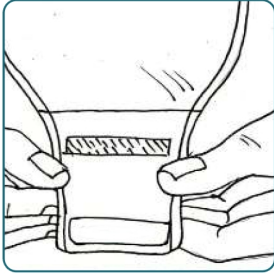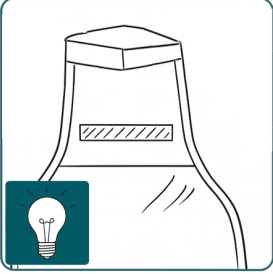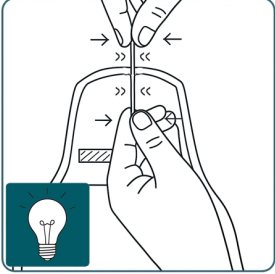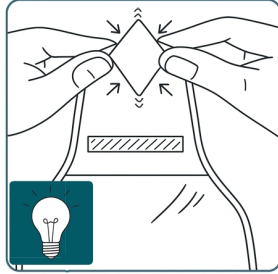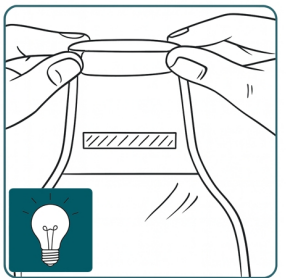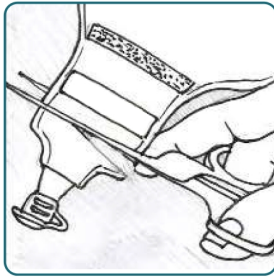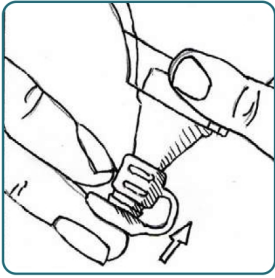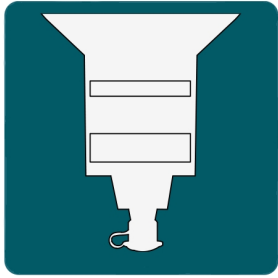

Intended Use, Product Description and Material Information

Ostomy pouches are used to collect waste from the body and are usually attached to the skin over a surgically made stoma. Zensiv Ostomy products are a range of 1-piece and 2-piece ostomy pouches and 2-piece flanges in various sizes and shapes. All 1-piece pouches and 2-piece flanges feature a skin friendly hydrocolloid wafer for adhesion to the body and this is protected before use with a release film that shows a cutting guide. The product pouch is manufactured from odour barrier film with a comfort non-woven layer and a carbon filter for odour control (for ileostomy and colostomy). Ileostomy pouches have either a roll up drain port secured with an integral 'velcro' type closure or there are clamps and soft ties supplied in each box. For urostomy the pouches carry a soft polymer tap for easy drainage. All pouches have a viewing window to inspect the stoma if needed. Some ranges feature a vent with lid for venting the pouch if required. All Zensiv Ostomy products are latex-free and do not contain ingredients of animal origin. The products are non-sterile and can be used in any appropriate circumstances where the attending physician or healthcare professional decides a non-sterile product may be applied.

Warnings-Precautions

- Each pouch is for single use only. Do not re-use. Re-use can lead to cross-contamination. - Use only as directed by your healthcare professional. - Read the instructions prior to use. - Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the pouches.

Contra-indications

These devices should not be used by anyone with a known sensitivity to: polyethylene, polyamide, medical charcoal, synthetic polymer, ethylene vinyl acetate, polypropylene, polyester, polyisobutylene, carboxymethylcellulose, pectin or glycerine, acrylic gel, spandex.

Storage Conditions

It is best to store your products in a cool dry place with a guidance of temperature from 0°C to 27°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf Life

Use by the expiry date noted on the label.

Collect your materials

Make sure that all necessary materials are ready for use. Collect a measuring guide, a pair of scissors, a pen, wet and dry gauze and a stoma pouch.

Choose the right size

Measure your stoma with a measuring guide and cut (if necessary) the flange to the right size of the stoma. Make sure that you choose the right size so that the stoma and surrounding skin will stay healthy. Allow a maximum of 2 mm of space between your stoma and flange for a safe and correct fit. If the size is too big, your faeces will damage the skin. If the size is too small, the flange will damage the stoma. Over time your stoma may change shape or size, so periodically measurements may be required again to prevent any leakage.

Prepare your skin

Support the skin and remove the flange carefully, do not tear off but push the skin back from the flange, use a wet gauze if necessary. Wash the skin surrounding your stoma with warm and unperfumed water. Do not use anything that contains alcohol, perfume or ether to clean your skin as it can cause damage. Do not rub the skin too hard as this might lead to skin damage. Dry the skin carefully and thoroughly to ensure that your pouch adheres well. Optional: If your skin is irritated and sore you can use skin barrier cream, hydrocolloid powder or skin barrier wipes. Apply the powder, skin barrier cream or wipe gently to the surrounding skin.

Close the tap at the base of the pouch

First make sure that the tap is closed safely. Close the tap by pressing the tap firmly.

Connecting your pouch to a night bag.

The pouch has a universal tap. This tap can be connected to almost any type and brand of night bag. Open the tap of the pouch and connect this to the connector of the night bag and ensure that the connection is secure.

Access window

To Open: Open the access window using the tab provided and pull open. Hold the pouch steady with your other hand while you do this. Ensure that the pouch is emptied before opening the access window. If this is not possible, hold the pouch carefully to ensure no output will spill out during the opening process. To Close: Close carefully by snapping the access window shut and pressing all the way round the closing section to ensure full closure. Always check that the access window is properly closed.

Fit your pouch

Remove the release paper of the flange. Fold the flange halfway and place the opening at the underside of the stoma. Stretch the skin of your abdomen with a flat hand and pull the whole flange over the stoma into the right position. Work your way around the stoma so that the hydrocolloid will attach to the skin and make sure that the flange is secure. Place your hand over the flange and make sure that a good seal is created.

Empty your pouch

Lift up the tap and open the tap. Empty the contents of your pouch into the toilet. Close tap again making sure it is securely closed.

Tap and outlet closure

If the tap is not required this can be cut off and the 'velcro' type closure used instead.

Empty your pouch using the 'velcro' type closure

Lift up the tail of the pouch and open the bag. Empty the contents of your pouch into the toilet.

Close the outlet

First make sure that the clamp or 'velcro' type closure is closed safely. For the clamp: Place the clamp at the tail of the pouch. Fold the end of the tail once over the clamp and make sure that the film of the tail is smooth to ensure that the outlet tail is in the right position and press the clamp firmly. Clamps are reusable. For the metal wire strips: Take the release film from the metal wire strip. Stick the metal wire strip onto the tail of the pouch. Roll-up the tail of the pouch. Fold the metal wire on both sides so that the tail of the pouch stays rolled-up. For the Velcro closure: Roll-up the tail of the pouch until the two 'velcro' type pieces are touching each other. Press the 'velcro' type pieces onto each other firmly. For the Velcro closure with folding tabs: Roll-up the tail of the pouch until the velcro is at the same height as the folding tabs. Press the velcro on the folding tabs firmly onto the velcro of the closure.

Disposal of your pouch

Remove your pouch and dispose of it in the trash. Do not flush your used pouch down the toilet because this will block your sewage system.

Frequency of pouch change

Change/empty your pouch as advised by your healthcare professional.

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