# Silfy Siliconen Tape

## GB: Silfy Silicone Tape

### Features and Intended Use

- Silfy Silicone tape is intended as a skin friendly and pain-free fixation device, particularly for sensitive skin.
- The tape retains its adhesion making it possible to remove it from the skin several times and re-apply it without the tape losing its adhesive strength.
- The Silfy Silicone tape is waterproof, will not absorb skin residues or move during wear.
- It can remain on the skin for several days if desired.
- The Silfy Silicone tape can be torn off the roll to any desired length.

### Intended use

- Silfy Silicone Tape is intended for skin friendly and pain free fixation to the skin. It is suitable for situations where the condition of the skin is fragile or damaged such as following radiotherapy treatment, chemotherapy or dialysis and can be used to give extra adhesion around the edges of wound management devices or stoma pouches. It can also be used with a variety of catheters or for other types of fixation that require good skin protection and strong adhesion.

### Materials

|           |   |
|-----------|---|
| Tape      | Polyethylene/Polyurethane film.   |
| Adhesive  | Silicone gel.   |
| Packaging | Tape is wound on a plastic core with protecting cap, labelled and placed in a white board box fully labelled to comply with CE marking regulations. |
| Roll size | 2.5cm wide x 1.5m length  |

### Tape Application, Removal and Disposal

- Before use, clean the skin with water and a mild soap. Avoid the use of creams and powders.
- Pat the skin well dry. Note: the tape does not adhere to a damp surface.
- Place the product to be fixed into the correct position.
- Determine the length of silicone tape required and tear off the tape to the correct length.
- Apply the tape to the desired area and press firmly for optimal adhesion.
- To remove – lift the end of the silicone tape and carefully remove from the skin.
- Dispose of the discarded tape in the household waste, never flush down the toilet.

### Frequency of Application

- If the tape stays in place there is no need to remove or replace it until desired.
- If the tape does lift from the skin or is no longer adhering well replace with fresh tape.

### Warnings/Precautions

- Read the instructions for use prior to use.
- Use only as directed by your healthcare professional.
- For external use only.
- Do not use with oxidizing solutions such as hypochlorite solution or hydrogen peroxide solution (H2O2).
- Discontinue use if any allergic reactions occur, such as red skin or itching, or problems are experienced when using the Silfy Silicone tape and, if required, contact a healthcare professional for advice.

### Storage Conditions

Store in a cool dry place with a guidance of temperature of 5°C to 30°C and humidity of less than 80% as is noted on the product label.

### Contraindications

- Silfy Silicone tape should not be used by people with a known sensitivity to any of the ingredients – see section Materials.
- Do not use on open wounds.

### Serious incidents

Should a serious incident arise as a result of using the Silfy Silicone tape, please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

### Shelf life

3 years. Use by the expiry date noted on the product label.

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