Hersubel Steunbandage Original

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Hersubel Range of Support Belts Original

Features and intended use

1. The Hersubel range of Support Belts are intended for use by ostomates who have a peristomal hernia, whose abdominal muscles post-surgery require extra support to prevent herniation, or who require extra support for undertaking sports activities or heavy work.
2. A Hersubel Belt will give the required support in the abdominal area by moulding over any shape or size of hernia and is comfortable and breathable during wear.
3. Hersubel Support Belts are provided in a range of widths and lengths to suit all users and are supplied in beige, white and black.
4. Manufactured from two way stretch fabric the Hersubel belt will reduce the bulges and dragging feelings created by herniation, whilst allowing the wearer ease of movement.
5. The comfort of the stretch fabric allows long wear times and the anti-roll silicone strips on the outer edges prevent unintended movement of the belt during wear or curling of the belt at the edges.
6. A Hersubel Belt is easy to apply over a stoma pouch using the integral glove and large Velcro® fixing pad. It will assist with pouch adhesion and create an even appearance, thereby increasing confidence. It must be noted that the part of the integral glove and large Velcro® fixing pad should never be placed over the stoma.
7. Hersubel Belts are washable in a washing machine using a low temperature wash – see section *Looking after your Hersubel Belt*.

Materials

1. Belt Materials Elastane and polyamide
2. Fixing pad Velcro®
3. Non slip strips Silicone
4. Packaging Individually packed in a board box fully labelled to comply with CE marking regulations.

Application of a Hersubel Support Belt

1. Remove the belt from the package and unfold .
2. Hold the belt with the gloved end in your hand and take the belt around behind you making sure that the silicone strips are facing your body so that they can prevent any slipping of the belt.
3. Ensure to place the belt comfortably and completely over your hernia and stoma bag. Be sure that the part with the integral glove and large Velcro® fixing pad are not placed over the stoma.
4. Hold the belt in place and with your hand in the glove, pull the belt to a comfortable tightness and adhere the gloved section to the large Velcro® fixing pad.
5. Removal of the belt – place your hand in the glove and pull the belt free from the Velcro® fixing pad.
6. Take the belt from around your body.

Looking after your Hersubel Support Belt

1. Hersubel Support Belts can be worn during the day and at night depending on the needs of the user.
2. The Hersubel Support Belt can be washed in the washing machine at a maximum of 40°C.
3. Hersubel Support Belts should be air dried and not dried in a tumble dryer as this can damage the material of the belt.

Discarding the Hersubel Support Belt

1. Throw away into a normal waste bin when the belt is no longer giving the correct support.

Warnings/precautions

1. Read the instructions for use prior to use.
2. Ensure that the belt is fitted around the body correctly and that the belt provides the pressure required for the ostomate (See section *Application of a Hersubel Support Belt*).
3. The Hersubel Support Belt should only be used on the recommendation of a healthcare professional.
4. The Hersubel Support Belt should be used only as a support belt.
5. Never place the integral glove/ large Velcro® fixing pad over the stoma.
6. Please ensure the washing instructions shown in section *Looking after your Hersubel Support Belt* are followed.
7. Discontinue use if you experience any allergic reaction or problems when using the Hersubel Support Belt and if required, contact a healthcare professional.
8. Length of time in use will depend on number of uses and amount of washing. The elastic materials of the belt will perform as any other corset or undergarments made of similar materials.

Storage conditions

1. No special storage conditions.

Contraindications

1. Do not use if you have a known allergy to any of the materials used in the belt.
2. Persistent abdominal pain while using the belt - contact a healthcare professional.
3. Poor manual dexterity or with visual impairment - do not use the belt without assistance

Serious incidents

Should a serious incident arise as a result of using the Hersubel Support Belt please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf life

5 years. Use by the expiry date noted on the product label.

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