Weldra Urine Bag 3000ml 140cm tube REF WMNZ3000

Features and intended use

1. The 3000ml capacity Weldra Urine Bag is for use as a urine collection bag and can be connected to most urostomy pouches and urine leg bag systems via the attached universal connector.
2. The Weldra Urine Bag is supplied non-sterile for use by individuals who require extra capacity for night time bladder drainage.
3. Included with each Weldra Urine Bag is a hanging frame with fastening cord, bed sheet clamp, and tube clamp.
4. The Weldra Urine Bag shows measurement graduations on one side of the bag, but these are for guidance only and are not precise measurements.
5. Emptying of the Weldra Urine Bag can be done using one hand via the emptying valve at the base of the bag.

Materials

- | | | |
|----|--|------------------------|
| 1. | Weldra Urine Bag and integral air vent: | Medical grade PVC film |
| 2. | Non return valve: | PVC |
| 3. | Kink resistant 140cm tubing: | Flexible PVC |
| 4. | Large bore connector: | PVC/LDPE |
| 5. | Valve outlet stowage: | PVC reinforced |
| 6. | Push type valve: | PVC |
| 7. | Bed sheet clamp & tube clamp: | ABS |
| 8. | Integral hanger: | Polypropylene |
| 9. | Packaging – each Night Drainage Bag is individually pouched with 10 pouches in a box | |

Application and disposal

Connecting the Weldra Urine Bag to a urostomy pouch or catheter.

1. At the base of the Weldra Urine bag, remove the outlet valve from its storage pocket. Make sure that this outlet valve at the bottom of the Weldra Urine bag is closed, if not then close it by moving the blue valve from left to right. Hang the Weldra Urine Bag on the hanging frame, next to the bed, using the fastening cord to secure it to the frame.
2. Remove the blue cap from the end of the connection tube that comes from the top of the bag and push the connector firmly into the opening of your urostomy pouch or catheter.
3. Check the connection is secure. Also, check the tube is straight and not kinked or twisted as this could cause a blockage in flow during the night. If desired, you can use the pinch clamp that is attached to the tube with the elastic band. This clamp will enable you to fix the tube in one place, for instance you can attach the clamp to your sleepwear so the tube cannot move back and forth during the night. A slide clamp is also attached to the tube for shutting off the flow of urine.
4. In the morning, disconnect the Weldra Urine bag from your catheter or urostomy pouch, and close the tap on the urostomy pouch. Empty the Weldra Urine bag into the toilet by opening the blue outlet valve at the bottom of the bag.
5. The Weldra Urine Bag is non-sterile and can be used more than once. Please ensure that the bag is well flushed out with water once per day.

Discarding the used Weldra Urine Bag

1. First empty the Weldra Urine Bag.
2. Flush out the Weldra Urine Bag and then place in a plastic (waste) bag or wrap it in a few layers of paper.
3. Throw the package in a waste bin.

Warnings/precautions

1. Read the instructions for use prior to use.
2. Ensure that all required connections have been snugly made and that the outlet valve is closed after emptying the Weldra Urine Bag.
3. The Weldra Urine Bag must never be flushed down the toilet.
4. The Weldra Urine Bag should be used only as a urine collection bag and on the recommendation of a healthcare professional.
5. Please ensure the cleaning instructions shown in point 5 of the APPLICATION AND DISPOSAL section are followed.
6. Discontinue use if you experience any allergic reaction or problems when using the Weldra Urine Bag and if required, contact a healthcare professional for advice.

Storage conditions

1. Store in a cool dry place.
2. Use Weldra Urine Bag immediately after opening each pouch.

Contraindications

1. Poor manual dexterity.
2. Poor visual capability.
3. Absence of trained caregiver.

Serious incidents

Should a serious incident arise as a result of using the non-sterile Weldra Urine Bag, please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf life

5 years. Use by the expiry date noted on the product label.

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