



# Zensetiv Stoma Belt

REF

ABS125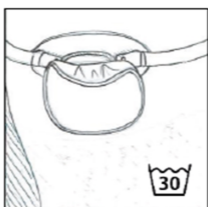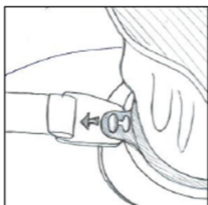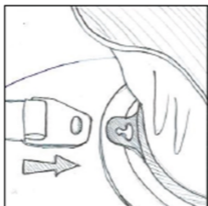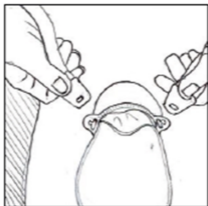

## GB - Instructions for use

### Features and Intended Use

The Zensetiv Stoma Belt is an elasticated belt used to provide additional security to assist in maintaining a stoma bag in place during wear. It has 2 attachment points that exert even pressure on the base plate. Belt size is 125cm long x 2.8 cm wide and is adjustable.

### Materials

Belt: 90% Cotton and 10% man-made fibre which is an elastic to facilitate the belt stretch.

Loops: ABS polymer (acrylonitrile butadiene styrene).

### Application

- Attach your ostomy appliance as usual.
- Then, passing the belt around your body, fasten each end of the belt by hooking the lugs of the belt into the eyelets on the belt holder of the ostomy appliance.
- The belt is individually adjustable to abdominal circumference using the adjustment loops.
- Make sure that the belt is not twisted.

### Disposal of the device after use

When the product cannot be re-used any more dispose of it in the trash.

### Warnings and Precautions

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.
- Discontinue use and contact your

healthcare professional if you experience any allergic reactions or problems when using the product.

- Incorrect adjustment to the abdominal circumference can lead to pressure points. Avoid adjusting too tightly.

- The belt can be washed at a low temperature in a laundry bag. A 30°C temperature is recommended.

### **Contraindications**

These devices should not be used by anyone with a known sensitivity to the materials or ingredients listed in the section 'Materials'.

### **Storage conditions**

No special storage conditions required.

### **Serious Incidents**

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

### **Shelf Life**

Use by the expiry date noted on the label.

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Oxmed International GmbH