


# Silicone Plaster

**REF**

**ASP275**

**ASP2100**

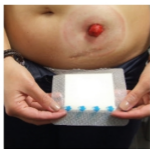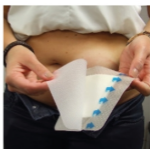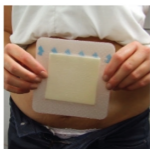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## GB - Instructions for use

### Features and Intended Use

Zensetiv Silicone Plasters are non-sterile single-use dressings intended to protect a stoma after irrigation or a continent stoma and may be used with fistulae where the attending physician decides a non-sterile dressing may be applied. The central island material of the plaster will absorb and retain secretion. The silicone-coated polyurethane foam adheres the plaster to the skin to prevent leakage and is non-irritant and easy to remove.

### Materials

A plaster of polyurethane film coated with silicone gel with an absorbent central island of polyurethane foam. Overlapping release film is adhered to the plaster for ease of application.

### Application

- Clean the skin around the stoma with water if necessary and dry the area well.- Avoid the use of alcohols, sprays or creams as this could affect the adhesion of the plaster. - Remove one side of the release film that covers the adhesive.
- Remove the other side of the release film and stick the plaster to the skin.
- Hold the plaster with the adhesive side towards your body.
- Apply the plaster over the stoma from bottom to top.
- Ensure to adhere it evenly to the body and smooth any creases that may occur.
- Secure the plaster by rubbing it gently.

## **Frequency of replacement of the plaster**

Change the plaster when leakage occurs, when the dressing is completely saturated, once a day or when advised by your healthcare professional.

## **Disposal of the device after use**

Remove the product and dispose of it in the trash. Do not flush your used product down the toilet because this will block your sewage system.

## **Warnings and Precautions**

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.
- Each product is for single use only. Do not re-use. Re-use can lead to cross-contamination.
- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.
- When using with a stoma - silicone plaster should only be used after irrigation of the stoma or when the stoma is known to be continent.
- Do not use while swimming or bathing.

## **Contraindications**

These devices should not be used by anyone with a known sensitivity to the materials or ingredients listed in the section 'Materials'.

## **Storage Conditions**

It is best to store your products in a cool dry place with a guidance of

temperature from 0°C to 27°C as is noted on the product label.

### **Serious Incidents**

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

### **Shelf Life**

Use by the expiry date noted on the label.

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Oxmed International GmbH