# Zensetiv Silicone Tape

**REF**

**AST25150**

**AST25150W**

## GB - Instructions for use

### Features and Intended Use

- Zensetiv Silicone tape is for skin friendly and pain-free fixation and it is extremely suitable for sensitive skin. The tape is painless to remove and retains its adhesion so that it can be removed from the skin several times and can be applied again without losing its adhesive strength.
- Strong and skin-friendly fixation.
- When removing the tape, the tape does not absorb skin residues and the skin remains intact.
- Waterproof.
- Easy to tear off at any desired length.
- Can stay on the skin for several days.
- Measurements: 2.5cm width x 1.5m length.

### Intended Use

- Zensetiv Silicone Tape is intended for skin friendly and pain free fixation to the skin. It is suitable for situations where the condition of the skin is fragile or damaged such as following radiotherapy treatment, chemotherapy or dialysis and can be used to give extra adhesion around the edges of wound management devices or stoma pouches. It can also be used with a variety of catheters or for other types of fixation that require good skin protection and strong adhesion.

### Materials

A polyurethane/polyethylene tape coated with silicone gel.

### Application

Application

- Before use clean the skin with water and your usual soap and dry well.

- Avoid the use of sprays, creams or powders as this can affect adhesion.
  - Put the product to be fixed in the correct position.
  - Determine the length of silicone tape required.
  - Tear/cut off the silicone tape to the correct length.
  - Apply the tape to the desired area and rub firmly for optimal adhesion.
- Removal
- Lift the end of the silicone tape.
  - Carefully remove the silicone tape.

### **Frequency of replacement of the product**

If the tape stays in place, there is no need to remove or replace it. If the tape lifts from the skin or is no longer adhering well, it should be replaced with some fresh tape.

### **Disposal of the device after use**

Remove the product and dispose of it in the trash. Do not flush your used product down the toilet because this will block your sewage system.

### **Warnings and Precautions**

- Use only as directed by your healthcare professional.
- Read the instructions prior to use.
- For external use only.
- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.
- Never apply to an open wound.

### **Contraindications**

These devices should not be used by anyone with a known sensitivity to the materials or ingredients listed in the

section 'Materials'.

### **Storage conditions**

Store in a cool dry place with a guidance of temperature of 5°C to 30°C and humidity of less than 80% as is noted on the product label.

### **Serious Incidents**

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

### **Shelf Life**

Use by the expiry date noted on the label.

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Oxmed International GmbH