It's all  
about  
ostomy!









GB	Instructions for use	NL	Gebruiksaanwijzing	DE	Gebrauchsanweisung
AR	تعليمات الاستخدام	BA	Uputstvo za upotrebu	CZ	Návod k použití
DK	Brugsanvisning	ES	Instrucciones de uso	FI	Käyttöohje
FR	Mode d'emploi	GR	Οδηγίες χρήσης	HR	Upute za uporabu
ID	Petunjuk Penggunaan	LT	Naudojimo instrukcijos	LV	Lietošanas instrukcija
NO	Bruksanvisning	PL	Instrukcja użycia	PT	Instruções de uso
RO	Instructiuni de folosire	SE	Bruksanvisning	SI	Navodila za uporabo
SK	Inštrukcie na používanie	TR	Kullanım talimatları	UA	Інструкція із застосування

## REF

U2B4F	U2B5F	U2B6F	U2B7F
U2C4F	U2C5F	U2C6F	U2C7F
U2Z4F	U2Z5F	U2Z6F	U2Z7F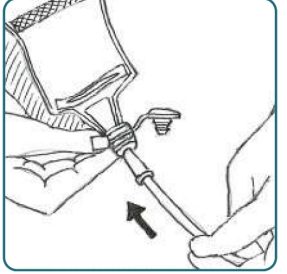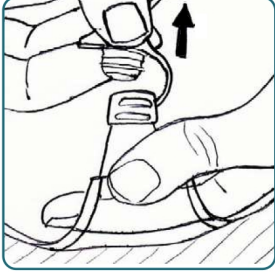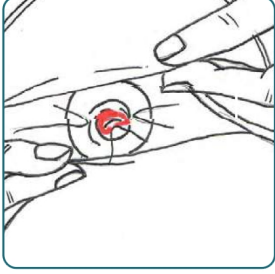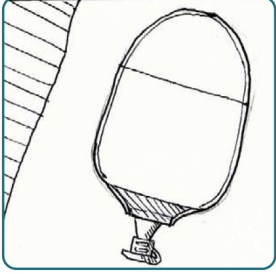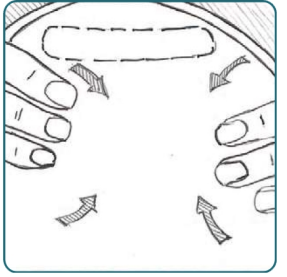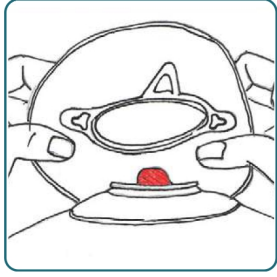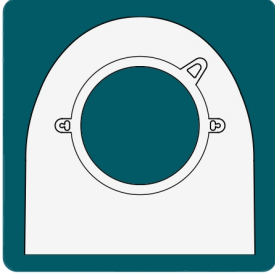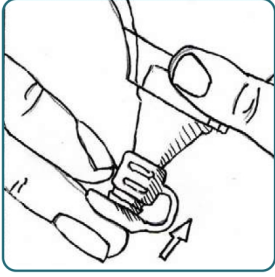

## GB - Instructions for use

### Intended Use, Product Description and Material Information

Ostomy pouches are used to collect waste from the body and are usually attached to the skin over a surgically made stoma. Zensiv Ostomy products are a range of 1-piece and 2-piece ostomy pouches and 2-piece flanges in various sizes and shapes. All 1-piece pouches and 2-piece flanges feature a skin friendly hydrocolloid wafer for adhesion to the body and this is protected before use with a release film that shows a cutting guide. The product pouch is manufactured from odour barrier film with a comfort non-woven layer and a carbon filter for odour control (for ileostomy and colostomy). Ileostomy pouches have either a roll up drain port secured with an integral 'velcro' type closure or there are clamps and soft ties supplied in each box. For urostomy the pouches carry a soft polymer tap for easy drainage. All pouches have a viewing window to inspect the stoma if needed. Some ranges feature a vent with lid for venting the pouch if required. All Zensiv Ostomy products are latex-free and do not contain ingredients of animal origin. The products are non-sterile and can be used in any appropriate circumstances where the attending physician or healthcare professional decides a non-sterile product may be applied.

### Warnings-Precautions

- Each pouch is for single use only. Do not re-use. Re-use can lead to cross-contamination. - Use only as directed by your healthcare professional. - Read the instructions prior to use. - Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the pouches.

### Contra-indications

These devices should not be used by anyone with a known sensitivity to: polyethylene, polyamide, medical charcoal, synthetic polymer, ethylene vinyl acetate, polypropylene, polyester, polyisobutylene, carboxymethylcellulose, pectin or glycerine, acrylic gel, spandex.

### Storage Conditions

It is best to store your products in a cool dry place with a guidance of temperature from 0°C to 27°C as is noted on the product label.

### Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

### Shelf Life

Use by the expiry date noted on the label.

### Collect your materials

Make sure that all necessary materials are ready for use. Collect a measuring guide, a pair of scissors, a pen, wet and dry gauze and a stoma pouch and flange.

### Choose the right size

Measure your stoma with a measuring guide and cut (if necessary) the flange to the right size of the stoma. Make sure that you choose the right size so that the stoma and surrounding skin will stay healthy. Allow a maximum of 2 mm of space between your stoma and flange for a safe and correct fit. If the size is too big, your faeces will damage the skin. If the size is too small, the flange will damage the stoma. Over time your stoma may change shape or size, so periodically measurements may be required again to prevent any leakage.

### Prepare your skin

Support the skin and remove the flange carefully, do not tear off but push the skin back from the flange, use a wet gauze if necessary. Wash the skin surrounding your stoma with warm and unperfumed water. Do not use anything that contains alcohol, perfume or ether to clean your skin as it can cause damage. Do not rub the skin too hard as this might lead to skin damage. Dry the skin carefully and thoroughly to ensure that your pouch adheres well. Optional: If your skin is irritated and sore you can use skin barrier cream, hydrocolloid powder or skin barrier wipes. Apply the powder, skin barrier cream or wipe gently to the surrounding skin.

### Close the tap at the base of the pouch

First make sure that the tap is closed safely. Close the tap by pressing the tap firmly.

### Connecting your pouch to a night bag.

The pouch has a universal tap. This tap can be connected to almost any type and brand of night bag. Open the tap of the pouch and connect this to the connector of the night bag and ensure that the connection is secure.

### Fit your base plate

Remove the release paper of the base plate. Stretch the skin of your abdomen with a flat hand and place the base plate over your stoma into the right position (start at the bottom and work your way to the top). Work your fingers around the base plate so that the hydrocolloid bonds well onto the skin and make sure that the plate is secure.

### Apply your pouch

Take your pouch and click the pouch onto the ring of base plate. Work your way from the bottom to the top of the ring. Make sure that your pouch is secure and control the complete seal of the pouch with your fingers.

### Empty your pouch

Lift up the tap and open the tap. Empty the contents of your pouch into the toilet. Close tap again making sure it is securely closed.

### Viewing window

The ostomy pouch includes a viewing window which makes it possible to inspect the stoma as well as the output from the stoma. It is also very helpful when applying the pouch to make sure it is located correctly over the stoma. Lift up the non-woven of the viewing window for inspection.

### Disposal of your pouch

Remove your pouch and dispose of it in the trash. Do not flush your used pouch down the toilet because this will block your sewage system.

### Frequency of pouch change

Change/empty your pouch as advised by your healthcare professional.

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