GB	Instructions for use	NL	Gebruiksaanwijzing	DE	Gebrauchsanweisung
AR	تعليمات الاستخدام	CZ	Návod k použití	DK	Brugsanvisning
FI	Käyttöohje	PL	Instrukcja użycia	PT	Instruções de uso
SE	Bruksanvisning	SK	Inštrukcie na používanie		

REF

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GB - Instructions for use

Please read these Instructions before use and follow the guidelines in the Method of use section.

Intended Use, Product Description and Material Information

The Zenso wound and fistula pouch is for use in draining and collecting output from wounds, stomas, particularly high output difficult to manage stomas and fistulae. Their use where post-operative wounds require drainage and nursing management following surgery allows the healthcare professional to cover the affected area while still being able to drain it of exudate, carry out other care or simply cover over the area during palliative care. The smaller sizes are for use where collection of exudate is required. Zenso wound and fistula pouch may be applied by healthcare professionals or by the patient/user following training by their healthcare professional. The Zenso wound and fistula pouch has no healing or treatment properties and is only intended for adhesion to intact skin. The Zenso wound and fistula pouch is a range of 1-piece pouches in various sizes and shapes. All pouches feature a skin friendly hydrocolloid wafer for adhesion to the body and this is protected before use with a release film that shows a cutting guide. Pouches are manufactured from odour barrier film with a comfort non-woven layer. All pouches carry a tap for easy drainage and most pouches have a roll up drain port secured with an integral 'velcro' type closure. Some pouches feature a vent with lid for venting the pouch or rinsing the wound area if required. All pouches have a transparent film to inspect the stoma if needed and some pouches have an access panel for nursing care. The Zenso wound and fistula pouch is latex-free and does not contain ingredients of animal origin.

Contra-indications

These devices should not be used by anyone with a known sensitivity to: polyethylene, polyamide, ethylene vinyl acetate, polypropylene, polyisobutylene, pectin, carboxymethylcellulose, glycerine, acrylic gel, spandex.

Method of use

- Use standard procedures for preparing the wound (or fistula) area. The skin surrounding the wound should be clean and dry. - Measure the wound and determine which pouch is most suitable. - Map a template of the wound area. This can be done by using an acetate or plain paper. - With curved scissors, cut the hydrocolloid wafer to match the size and shape of the template. Do not cut outside the indicated area. The cut out pieces of hydrocolloid can be used for extra protection or packing, particularly around the output area of the wound. - If necessary, use additional ostomy products to protect the skin around the wound from output. - Remove the release paper and position the pouch over the wound. Smooth into position around the wound; applying gentle pressure. Maintaining this gentle pressure on the hydrocolloid for approximately 30 seconds has the effect of maximizing adhesion. - The tap should be closed until a collection device is required. For drainage, plug the connector from the collection device into the tap until a tight seal is achieved. - If the pouch contains a 'velcro' and the tap is not used, there is an option to cut off the tap. After cutting off the tap, close the 'velcro'. - On pouches that feature an access panel, access can be gained to the wound area by opening the access panel. Take care to ensure the access panel is fully closed when finished. This will avoid inadvertent leakage and / or odour. Ensure the access panel is clean for secure closure. - When removing pouch do so carefully using the side of the thumb by working from the outside edge of the hydrocolloid wafer pushing towards the centre all the way round and ensure the skin is not pulled upwards during removal. Use a remover spray if necessary to remove any residue.

Frequency of pouch change

The interval between changes will normally be determined by the degree of exudates produced. The pouch may be left in place for up to 7 days.

Disposal

Following removal dispose of the pouch in hospital contaminated waste or local contaminated waste care disposal.

Cautions/Warning

- The pouches are non-sterile and can be used in any appropriate circumstances where the attending physician or healthcare professional decides a non-sterile pouch may be applied. - The pouches may be applied by healthcare professionals or by the patient/user following training by their healthcare professional. - For single use only. Never re-use this device because cross contamination can occur. - Discontinue use if any allergic reactions occur. - If used by an individual – if problems occur with the use of this device contact a healthcare professional for advice. - The pouch should be adhered to intact skin and not be used on exposed muscle or bone, so must be cut to fit on the available skin surrounding the wound.

Storage conditions

Store between 0°C to 27°C in a cool dry place.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf Life

Use by the expiry date noted on the product label.

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