Zensetiv Adhesive Remover Spray

REF

ZSA2011

Keep out of reach of children.

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GB - Instructions for use

Features and Intended Use

Zensetiv Adhesive Remover Spray is intended to aid the removal of medical adhesives (for example wound dressings, tapes or stoma flanges) and/or residues of medical adhesives from intact skin.

Materials

Hexamethyldisiloxane, butane, propane, isobutane.

Application

When removing adhesive dressings:

- Lift up the edge of the medical adhesive and apply Zensetiv Adhesive Remover under the product while lifting off.
- Peel away the product slowly while continuing to apply more adhesive remover as the product begins to lift.
- Let the skin dry and a new product can be applied if required.

For removing adhesive residue on the skin:

- Apply adhesive remover to the affected area.
- Re-apply for extremely difficult residues.

Disposal of the device after use

Dispose of contents/container in accordance with local regulations.

Warnings-Precautions

- Read the instructions prior to use.
- Use in a well ventilated area.
- Danger: Extremely flammable aerosol. Pressurised container: May burst if heated.
- Toxic to aquatic life with long lasting

effects.

- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Do not spray on an open flame or other ignition source.
- Do not pierce or burn, even after use.
- Protect from sunlight. Do not expose to temperatures exceeding 50°C/ 122°F.
- Avoid release to the environment.
- Collect spillage.
- Keep out of reach of children.
- Discontinue use and contact your healthcare professional if you experience any allergic reactions or problems when using the product.

Contraindications

These devices should not be used by anyone with a known sensitivity to the materials or ingredients listed in the section 'Materials'.

Zensetiv Adhesive Remover should not be used on infected or breached skin.

Avoid use on sensitive areas of the body (eyes, ears, nose, mouth, genitalia).

Storage conditions

Keep in a cool, dry, well ventilated place out of direct sunlight. Store between 5°C and 30°C as is noted on the product label.

Serious Incidents

Should a serious incident arise as a result of using the product please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf life

Use by expiry date printed on the base of the container.

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Oxmed International GmbH