Hersubel Tube

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Hersubel Tube

Features and intended use

1. The Hersubel range of Support Belts are intended for use by ostomates who have a peristomal hernia, whose abdominal muscles post-surgery require extra support to prevent herniation, or who require extra support for undertaking sports activities or heavy work.
2. The Hersubel Tube can also be used for leisure activities and light work and give support when coughing and sneezing.
3. The Hersubel Tube will give the required support in the abdominal area by moulding over any shape or size of hernia and is comfortable and breathable during wear.
4. The Hersubel Tubes are provided in a range of widths and lengths to suit all users and are supplied in beige and black.
5. Manufactured from two way stretch fabric the Hersubel Tube will reduce the bulges and dragging feelings created by herniation, whilst allowing the wearer ease of movement.
6. The comfort of the stretch fabric allows long wear times and the anti-roll silicone strips on the outer edges prevent unintended movement of the Hersubel Tube during wear or curling of the Tube at the edges.

Materials

- | | |
|--------------------|--|
| 1. Tube Materials | Cotton and Elastane |
| 2. Non slip strips | Silicone |
| 3. Packaging | Individually packed in a board box fully labelled to comply with CE marking regulations. |

Application of the Hersubel Tube

1. The Hersubel Tube can be applied standing up or horizontally. For more information see the photos of the Hersubel Tube standing up application, further down in the Instruction For Use. The text with the photo's also applies when applying the Hersubel Tube horizontally.
2. Removal of the Hersubel Tube; place your hands on the inside of the Hersubel Tube. Gently push the Tube forward and down. Pull the Tube further down to your feet.

Looking after your Hersubel Tube

1. The Hersubel Tube can be washed in the washing machine at a maximum temperature of 40°C, using a detergent without bleach.
2. The Hersubel Tube should be air dried and not dried in a tumble dryer as this may damage the material of the Tube.
3. Wash the Hersubel tube immediately after contact with ostomy output.

Discarding the Hersubel Tube

1. Throw away into a normal waste bin when the Tube is no longer giving the correct support.

Warnings/precautions

1. Read the instructions for use prior to use.
2. The Hersubel Tube should only be used on the recommendation of a healthcare professional.
3. The Hersubel Tube should be used only as a support Tube.
4. Make sure that the Hersubel Tube fits properly and provides the pressure necessary for the stoma wearer.
5. Please ensure the washing instructions shown in section Looking after your Hersubel Tube are followed.
6. Discontinue use if you experience any allergic reaction or problems when using the Hersubel Tube and if required, contact a healthcare professional.
7. Length of time in use will depend on number of uses and amount of washing. The elastic materials of the Tube will perform as any other corset or undergarments made of similar materials.
8. When the Hersubel Tube no longer provides the correct support, it is recommended to have the size reassessed.

Storage conditions

1. No special storage conditions.

Contraindications

1. Do not use If you have a known allergy to any of the materials used in the Tube.
2. Persistent abdominal pain while using the Tube - contact a healthcare professional.
3. Poor manual dexterity or with visual impairment - do not use the Tube without assistance

Serious incidents

Should a serious incident arise as a result of using the Hersubel Tube please report this immediately to the manufacturer and the Competent Authority in the country where you reside.

Shelf life

5 years. Use by the expiry date noted on the product label

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